



Hypafix®

wide-area dressing fixation

White non-woven polyester, coated with skin-friendly polyacrylate adhesive.



For the fixation of large wound dressings, especially on frequently moved and highly contoured parts of the body. The dressing is firmly secured on all sides and cannot slip or become detached.



Easy application due to the integrated split liner.

Simple, individual, and economical

Available in five different widths, just cut to the required length. This saves material and avoids unnecessary wastage.



Skin-friendly and safe

The polyacrylate adhesive holds dressings safely and reliably in place.

The soft, stretchable material adapts well to body contours without causing constriction or congestion. The wound dressing is protected all around against contamination.

- Air- and moisture-permeable - reduces the risk of macerations.
- Helps protect wound dressing from contamination.
- Radiotransparent - does not need to be removed either for radiotherapy or X-ray examinations.

Leukoplast®

Wound care in best hands


Wide-Area Dressing Fixation with Hypafix®

General Information

Carefully clean and dry the surrounding skin. Ensure that the wound dressing is covered completely and the adhesive fixation edge is broad enough. Broader distal and proximal fixation edges should be used on parts of the body that are subject to stress and frequently moved.

Do not apply Hypafix® under tension and avoid creating skinfolds. Hypafix® is transversely stretchable and the direction of stretch thus corresponds to the direction of movement of the area to be treated.

Always ensure an adequate fixation edge when cutting to size!

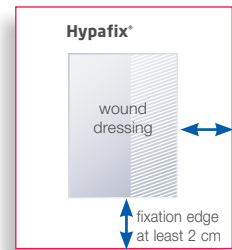

1. Cut Hypafix® to size.



2. Open the split liner by pulling slightly apart.



3. Remove the narrow part of the release paper ...



4. ... and apply the fixation edge on the surrounding skin.



5. Fold back the remaining part. Position the wound dressing as required and fix with the adhesive edge.



TWO WAYS OF CONTINUING ...



6. Pull the release paper with Hypafix® over the wound dressing...



...or push the release paper with Hypafix® over the wound dressing.

FINISHED!

Wide-area dressing fixation with HYPAFIX®.



Hypafix®		
Size	BNR	UOM/Case
2 in × 11 yds / 5 cm × 10 m	7144301	24
4 in × 11 yds / 10 cm × 10 m	7144302	24
6 in × 11 yds / 15 cm × 10 m	7144303	24
2 in × 2 yds / 5 cm × 2 m	7193921	36
4 in × 2 yds / 10 cm × 2 m	7193919	12
6 in × 2 yds / 15 cm × 2 m	7193920	12



NOTE: Please refer to the product label and/or package insert for full instructions on safe use of these products.